**THUASNE®**

# Ortel C1 Anatomic

<b>fr</b>	Collier cervical anatomique souple.....	5
<b>en</b>	Soft anatomical cervical collar.....	6
<b>de</b>	Weiche anatomische Halskrause.....	7
<b>nl</b>	Flexibele anatomische nekbrace.....	9
<b>it</b>	Collare cervicale anatomico morbido.....	10
<b>es</b>	Collarín cervical anatómico flexible.....	11
<b>pt</b>	Colar cervical anatómico flexível.....	13
<b>da</b>	Anatomisk, smidig nakkekrave.....	14
<b>fi</b>	Taipuisa anatominen kaulatuki.....	15
<b>sv</b>	Mjuk anatomisk nackkrage.....	17
<b>el</b>	Αυχενικό κολάρο ανατομικό μαλακό.....	18
<b>cs</b>	Anatomický měkký krční límec.....	19
<b>pl</b>	Miękki kołnierz anatomiczny do unieruchomienia szyi.....	20
<b>lv</b>	Anatomiski mīksta kakla ortoze.....	22
<b>lt</b>	Lankstus anatominis kaklo įtvėras.....	23
<b>et</b>	Pehme anatoomiline krae.....	24
<b>sl</b>	Opornica za vrat, anatomska, prožna.....	26
<b>sk</b>	Anatomický ohybný krčný golier.....	27
<b>hu</b>	Lágy, anatómiai kialakítású nyakmerevítő.....	28
<b>bg</b>	Мека анатомична цервикална яка.....	30
<b>ro</b>	Guler cervical anatomic flexibil.....	31
<b>ru</b>	Мягкий анатомический шейный воротник.....	32
<b>hr</b>	Mekana anatomska ortoza za vratnu kralježnicu.....	34
<b>zh</b>	符合解剖形状的软质颈圈.....	35
<b>ar</b>	طوق للعنق متوافق مع شكل العنق ومرن.....	37

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Istruțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	a	b			
0	24 - 29 cm	6 cm	-	-	-
1	28 - 33 cm		7,5 cm	9 cm	11 cm
2	34 - 39 cm				
3	40 - 46 cm	-			

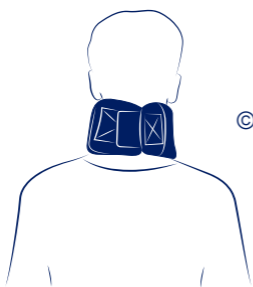
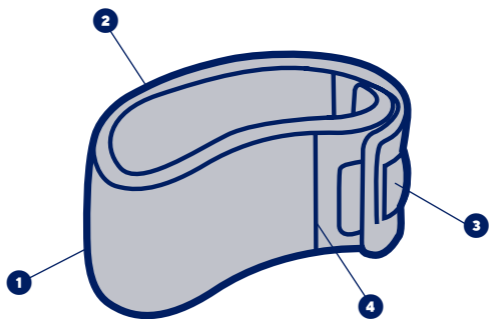
<b>fr</b>	(a) Tour de cou (b) Hauteur de cou	<b>lv</b>	(a) Kakla apkārtmērs (b) Kakla augstums
<b>en</b>	(a) Neck circumference (b) Neck height	<b>lt</b>	(a) Kaklo apimtis (b) Kaklo aukštis
<b>de</b>	(a) Halsumfang (b) Halshöhe	<b>et</b>	(a) Kaela ümbermõõt (b) Kaela pikkus
<b>nl</b>	(a) Omtrek van de nek (b) Hoogte van de nek	<b>sl</b>	(a) Obseg vratu (b) Dolžina vratu
<b>it</b>	(a) Circonferenza del collo (b) Altezza del collo	<b>sk</b>	(a) Obvod krku (b) Výška krku
<b>es</b>	(a) Circunferencia del cuello (b) Altura del cuello	<b>hu</b>	(a) Nyak kerülete (b) Nyak magassága
<b>pt</b>	(a) Perímetro do pescoço (b) Altura do pescoço	<b>bg</b>	(a) Размер на яката (b) Височина на яката
<b>da</b>	(a) Halsens omkreds (b) Halsens højde	<b>ro</b>	(a) Circumferința gâtului (b) Înălțimea gâtului
<b>fi</b>	(a) Kaulanympärys (b) Kaulan korkeus	<b>ru</b>	(a) Обхват шеи (b) Высота шеи
<b>sv</b>	(a) Runt halsen (b) I höjd med halsen	<b>hr</b>	(a) Opseg vrata (b) Visina vrata
<b>el</b>	(a) Περίμετρος λαιμού (b) Ύψος λαιμού	<b>zh</b>	( a ) 颈围 ( b ) 颈高
<b>cs</b>	(a) Obvod krku (b) Výška krku	<b>ar</b>	محيط العنق (a) ارتفاع العنق (b)
<b>pl</b>	(a) Obwód szyi (b) Wysokość szyi		

<b>fr</b>	Maintien	Effet proprioceptif
<b>en</b>	Support	Proprioceptive effect
<b>de</b>	Halt	Propriozeptive Wirkung
<b>nl</b>	Fixatie	Proprioceptieve werking
<b>it</b>	Sostegno	Effetto propioceettivo
<b>es</b>	Sujeción	Efecto propioceptivo
<b>pt</b>	Manutenção	Efeito propioceptivo
<b>da</b>	Støtte	Proprioceptisk effekt
<b>fi</b>	Tuki	Liikeaistiin kohdistuva vaikutus
<b>sv</b>	Stöd	Proprioceptiv effekt
<b>el</b>	Συγκράτηση	Αποτέλεσμα ιδιοδεκτικότητας
<b>cs</b>	Stabilizace	Proprioceptivní účinek
<b>pl</b>	Podtrzymywanie	Propriocepcja
<b>lv</b>	Atbalsts	Proprioceptīvs efekts
<b>lt</b>	Palaikymas	Propriocepčinis poveikis
<b>et</b>	Hooldus	Proprioceptiivne toime
<b>sl</b>	Oprijem	Proprioceptivni učinek
<b>sk</b>	Podpora	Proprioceptívny účinok
<b>hu</b>	Támasz	Proprioceptív hatás
<b>bg</b>	Поддръжка	Проприоцептивен ефект
<b>ro</b>	Susținere	Efect proprioceptiv
<b>ru</b>	Стабилизация	Проприоцептивный эффект
<b>hr</b>	Potpora	Učinak proprioceptivnog vježbanja
<b>zh</b>	穩定	本体感
<b>ar</b>	الدعم	مفعول استقبال الحس العميق

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### **Description/Destination**

Anatomically-shaped support foam. ①

Rounded edges for appropriate comfort. ②

Precise self-fastening adjustment. ③

Convenient removable cover. ④

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Cotton - elastane - polyamide - polyester - polyurethane.

### **Properties/Mode of action**

Support.

Proprioceptive effect.

### **Indications**

Cervical arthritis.

Cervical syndrome.

Dropped head syndrome.

Mild whiplash/mild sprain.

Post-operative use.

Rehabilitation after wearing a rigid cervical collar.

Torticollis (stiff neck).

### **Contraindications**

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

### **Precautions**

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the device in a medical imaging machine.

Do not wear the device while sleeping.

Do not wear the product more than 10 days without your healthcare professional's recommendation.

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

In the event of long-term wear of the product, possible secondary risks related to muscle atrophy.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Open the self-fastener.

Place the collar around the neck **A**.

The chin should rest on the chin support.

Close the self-fastener at the nape of the neck **B**.

Ensure the strap is not fastened too tight **C**.

### **Care/Maintenance**

See product label. Close the self-fastening tabs before washing. Hand wash.

Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

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Pazite da traku © ne stegnete previše.
Thuasne SAS - SIREN/RCS Nanterre 542 091 186