
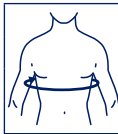
THUASNE



Cemen

fr	Ceinture thoracique.....	6
en	Thoracic belt.....	7
de	Brustorthese.....	8
nl	Borstkasgordel.....	9
it	Cintura toracica.....	10
es	Faja torácica.....	12
pt	Cinta torácica.....	13
da	Brystbælte.....	14
fi	Tukivyö ylävartalolle.....	15
sv	Bälte för bröstkorgen.....	16
el	Ζώνη θωρακικής χώρας.....	17
cs	Hrudní pás.....	19
pl	Orteza piersiowa.....	20
lv	Krūškurvja josta.....	21
lt	Torakalinis diržas.....	22
et	Rindkere vöö.....	23
sl	Oporni pas za prsni koš.....	24
sk	Hrudný pás.....	25
hu	Mellkasi öv.....	27
bg	Колан на гърдите.....	28
ro	Centură toracică.....	29
ru	Грудопоясничный корсет.....	30
hr	Torakalni pojas.....	31
zh	胸帶.....	32
ar	حزام للصدر.....	35


	1	60 - 75 cm
	2	76 - 90 cm
	3	91 - 105 cm
	4	106 - 120 cm

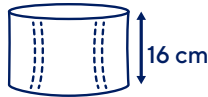
fr	Circonférence sous poitrine	Hauteur : 16 cm
en	Chest measurement	Height: 16 cm
de	Umfang unter der Brust	Höhe: 16 cm
nl	Omtrek onder borstkas	Hoogte: 16 cm
it	Circonferenza sotto il petto	Altezza: 16 cm
es	Circunferencia bajo el pecho	Altura: 16 cm
pt	Circunferência sob o peito	Altura: 16 cm
da	Omkreds under bryst	Højde: 16 cm
fi	Ympärymitta rinnan alapuolella	Korkeus: 16 cm
sv	Omkrets under bröstet	Höjd: 16 cm
el	Περιμετρος κάτω από το στήθος	Ύψος: 16 εκ.
cs	Obvod pod hrudníkem	Výška: 16 cm
pl	Obwód poniżej piersi	Wysokość: 16 cm
lv	Apkārtmērs zem krūtīm	Augstums: 16 cm
lt	Krūtinės apimtis	Aukštis: 16 cm
et	Rinnaalune ümbermõõt	Kõrgus: 16 cm
sl	Obseg pod prsmi	Višina: 16 cm
sk	Obvod hrudníka	Výška: 16 cm
hu	Körméret a mell alatt	Magasság: 16 cm
bg	Обиколка под гърдите	Височина: 16 cm
ro	Circumferința sub piept	Înălțime: 16 cm
ru	Обхват под грудью	Высота: 16 см
hr	Opseg ispod grudi	Visina: 16 cm
zh	胸下围	高度 : 16 cm
ar	محيط أسفل الصدر	الارتفاع: 16 سم

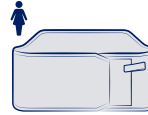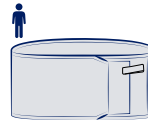
Homme - Femme	Compression	Soutien
Men - Women	Compression	Support
Männer - Damen	Kompression	Stütze
Heren - Dames	Compressie	Ondersteuning
Uomo - Donna	Compressione	Sostegno
Hombre - Mujer	Compresión	Sujeción
Homem - Mulher	Compressão	Suporte
Mand - Kvinde	Kompression	Støtte
Miehet - Naiset	Kompressio	Tuki
Herr - Dam	Kompression	Stöd
Ανδρας - Γυναίκα	Συμπίεση	Υποστήριξη
Muž - Žena	Kompresje	Podpora
Mężkie - Damskie	Kompresja	Podtrzymywanie
Vīriešu - sieviešu	Kompresija	Atbalsts
Vyri - Moterys	Kompresija	Palaikymas
Mees - Naine	Kompressioon	Toetus
Moški - Ženski	Kompresija	Podpora
Muž - Žena	Kompresija	Podpora
Férfiak - Nők	Szorítás	Megtámasztás
Мъжки модел - Женски модел	Компресия	Поддръжка
Bărbați - Femei	Compresie	Susținere
Мужчины - Женщины	Компрессия	Поддержка
Muškarci - Žene	Kompresija	Potpورا
男-女	压紧	支撑
للرجال - النساء	المصط	الدعم

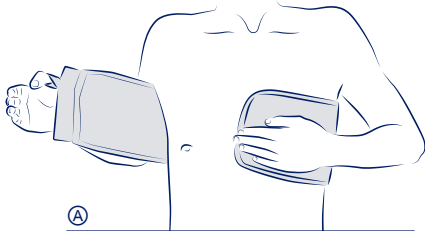A

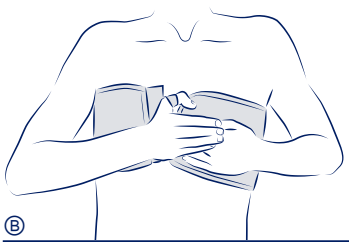B

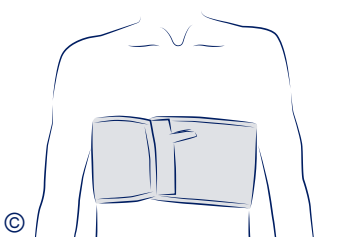C

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THORACIC BELT

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Compact elastic fabric. ❶

Closure with self-fastening surface and thumb-loop. ❷

Textile components: cotton - elastane - polyamide - polyester - polyurethane - viscose.

Non-textile components: polyamide.

Properties/Mode of action

The device ensures strong support and adaptable compression to the rib cage thanks to the elastic compact fabric.

Indications

Rib fracture.

Thoracic trauma/injuries.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use on pregnant women.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

The use of a medical device by a child, should be done under the supervision of an adult or a healthcare professional.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Position the belt around the chest, using the thumb-loop. ④

The inside label must be positioned upwards.

Hold the left end flat using your left hand.

Pull the thumb-loop simultaneously and evenly. ⑤

Fasten the belt, right side over left side, adjusting to the desired compression. ⑥

If the self-fastening strap is properly positioned it does not wrinkle.

The level of compression can be adjusted as required throughout the day.

Care

Close the self-fastening tabs before washing. Hand wash. Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Puxar simultaneamente e de modo homogêneo o espaço para os polegares. ©
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros