# THUASNE

# SPORT

<b>fr</b>	<b>Chevillère de maintien renforcée Thuasne Sport.....</b>	<b>3</b>
<b>en</b>	<b>Thuasne Sport reinforced ankle support .....</b>	<b>4</b>
<b>de</b>	<b>Verstärkte Knöchelbandage Thuasne Sport.....</b>	<b>5</b>
<b>nl</b>	<b>Thuasne Sport verstevigde steunbrace voor de enkel .....</b>	<b>7</b>
<b>it</b>	<b>Cavigliera di sostegno rinforzata Thuasne Sport .....</b>	<b>8</b>
<b>es</b>	<b>Tobillera de sujeción reforzada Thuasne Sport .....</b>	<b>9</b>
<b>pt</b>	<b>Tornozeleira de suporte reforçada Thuasne Sport.....</b>	<b>11</b>
<b>da</b>	<b>Forstærket ankelstøtte Thuasne Sport .....</b>	<b>12</b>
<b>fi</b>	<b>Vahvistettu Thuasne Sport -nilkkatuki .....</b>	<b>13</b>
<b>sv</b>	<b>Thuasne Sport förstärkt ankelortos för stöd.....</b>	<b>15</b>
<b>el</b>	<b>Ενισχυμένη επιστραγαλίδα συγκράτησης Thuasne Sport.....</b>	<b>16</b>
<b>cs</b>	<b>Zesílená kotníková bandáž Thuasne Sport.....</b>	<b>17</b>
<b>pl</b>	<b>Wzmocniona podtrzymująca orteza skokowo-stopowa Thuasne Sport .....</b>	<b>19</b>
<b>lv</b>	<b>Thuasne Sport potītes atbalsts ar pastiprinājumu .....</b>	<b>20</b>
<b>lt</b>	<b>Sutvirtintas palaikomasis kulkšnies įtvaras „Thuasne Sport“ .....</b>	<b>21</b>
<b>et</b>	<b>Thuasne Sport tugevdatud pahklutugi .....</b>	<b>22</b>
<b>sl</b>	<b>Ojačana opornica za gleženj Thuasne Sport .....</b>	<b>24</b>
<b>sk</b>	<b>Vystužená ortéza členku Thuasne Sport.....</b>	<b>25</b>
<b>hu</b>	<b>Thuasne Sport megerősített bokarögzítő .....</b>	<b>26</b>
<b>bg</b>	<b>Наглезенник с подсилена опора Thuasne Sport .....</b>	<b>28</b>
<b>ro</b>	<b>Glezneră de susținere întărită Thuasne Sport .....</b>	<b>29</b>
<b>ru</b>	<b>Поддерживающий голеностопный ортез Thuasne Sport с усиленной защитой .....</b>	<b>30</b>
<b>hr</b>	<b>Ojačani potporni steznik za gležanj Thuasne Sport... </b>	<b>32</b>
<b>zh</b>	<b>Thuasne Sport加强型稳定护踝.....</b>	<b>33</b>
<b>ar</b>	<b>Thuasne Sport دعامة مسك الكاحل المُعزّزة.....</b>	<b>35</b>

 cm	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>	<b>XXL</b>
	19 - 21	21 - 23	23 - 25	25 - 27	27 - 30

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---

en

## THUASNE SPORT REINFORCED ANKLE SUPPORT

### Description/purpose

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### Composition

Polyamide - Elastodiene - Elastane - Viscose.

### Properties/Mode of action

Elastic compressive knitting.

Comfort zone over the ankle bones.

### Indications

Joint instability/laxity.

Proprioceptive support for rehabilitation.

Mild ankle sprain (acute phase).

Sport activities resumption.

### Contraindications

Do not apply the product in direct contact with broken skin or an open wound without an adequate dressing.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

### Precautions

Read the instruction leaflet carefully.

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Consult a healthcare professional:

- In case of doubt before use,

- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,

- In case of changes in the device's performance.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

### Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity. Possible risk of venous thrombosis.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### Instructions for use/Application

Pull the ankle brace over the ankle joint, like a sock.

Place the malleoli correctly, on the comfort zones.

### Care

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Hand wash. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

### Storage

Store at room temperature, preferably in the original packaging.

### Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2033301 (2022-02)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186