THUASNE

SPORT

- fr Coudière de maintien Thuasne Sport
- en Thuasne Sport Elbow support
- de Ellenbogenbandage Thuasne Sport
- nl Thuasne Sport steunbrace voor de elleboog
- it Gomitiera di sostegno Thuasne Sport
- es Codera de sujeción Thuasne Sport
- pt Cotoveleira de manutenção Thuasne Sport
- da Albuebind Thuasne Sport
- fi Thuasne Sport -kynärpäätuki
- sv Armbågsstöd Thuasne Sport
- el Περιαγκωνίδα συγκράτησης Thuasne Sport
- cs Loketní bandáž Thuasne Sport
- pl Opaska podtrzymująca łokieć Thuasne Sport
- lv Elkoņa atbalsts Thuasne Sport
- lt Palaikomasis alkūnės įtvaras "Thuasne Sport"
- et Thuasne Sport küünarnukiortoos
- sl Opornica za komolec Thuasne Sport
- sk Bandáž na lakeť Thuasne Sport
- hu Könyökrögzítő Thuasne Sport
- bg Поддържащ налакътник Thuasne Sport
- ro Cotieră de susținere Thuasne Sport
- ru Поддерживающий налокотник Thuasne Sport
- hr Steznik za potporu Thuasne Sport
- zh Thuasne Sport 稳定护肘
- ar Thuasne Sport داعم مرفق

	S	22 - 24 cm
	M	25 - 27 cm
	L	28 - 30 cm
	XL	31 - 33 cm

fr	Tour de l'avant-bras
en	Forearm circumference
de	Umfang des Unterarms
nl	Omtrek van de voorarm
it	Circonferenza dell'avambraccio
es	Circunferencia del antebrazo
pt	Perímetro do antebraço
da	Omkreds af underarm
fi	Yläkäsivarren ympärysmitta
sv	Runt underarmen
el	Περίμετρος του αντιβραχίου
cs	Pro předloktí
pl	Opaska na przedramię
lv	Apkārt apakšdelmam
lt	Dilbio apimtis
et	Küünarvarre ümbermõõt
sl	Obseg podlakti
sk	Obvod predlakti
hu	Alkar kerülete
bg	Обиколка на предмишницата
ro	Circumferința antebrățului
ru	Обхват предплечья
hr	Opseg podlaktice
zh	前臂周长
ar	

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THUASNE SPORT ELBOW SUPPORT

Description/Use:

This device is an elbow brace providing joint support and stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition:

Textile components: polyamide - elastane - cotton.

Properties/Mode of action:

Even compression on the joint.

Improved proprioception.

Indications:

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Support for physical activity resumption.

Contraindications:

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Precautions:

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches..).

Undesirable side-effects:

This device can cause skin reactions (redness, itching, burns, blisters..) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application:

Pull the product on (the logo indicates the top and front of the product).

Make sure the elbow is properly centred in the zone provided for this purpose.

Care:

Product can be washed in accordance with the conditions shown on this notice and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

Storage:

Store at room temperature, preferably in the original packaging.

Disposal:

Dispose of in accordance with local regulations.

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©Thuasne - 2025701 (2021-04)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186