THUASNE SPORT

- fr Genouillère ligamentaire renforcée Thuasne Sport
- en Thuasne Sport reinforced ligament knee brace
- de Verstärkte Knieführungsorthese Thuasne Sport
- nl Thuasne Sport versterigde ligamentaire kniebrace
- it Ginocchiera per legamenti rinforzata Thuasne Sport
- es Rodillera ligamentaria reforzada Thuasne Sport
- pt Joelheira ligamentar reforçada Thuasne Sport
- da Forstærket knæbind til ledbånd Thuasne Sport
- fi Vahvistettu Thuasne Sport -polvituki nivelsiteiden hoitoon
- sv Thuasne Sport förstärkt knästöd för ledband
- el Ενισχυμένη επιγονατίδα συνδέσμων Thuasne Sport
- cs Zpevněná ortéza na kolenní vazy Thuasne Sport
- pl Opaska podtrzymująca więzadła kolanowe, wzmocniona Thuasne Sport
- lv Ceļa locītavas saišu ortoze ar pastiprinājumu Thuasne Sport
- lt Sutvirtintas raiščių antkelis "Thuasne Sport"
- et Põlveliigese sidemeid toetav tugevdatud põlvetugi Thuasne Sport
- sl Opornica za kolenske vezi Thuasne Sport
- sk Vystužená podpora kolenných väzov Thuasne Sport
- hu Megerősített térdrögzitő Thuasne Sport
- bg Подсилена лигаментна наколенка Thuasne Sport
- ro Genunchieră de susținere ligamentară ranforsată Thuasne Sport
- ru Усиленный коленный ортез Thuasne Sport
- hr Ojačani steznik za ligamente koljena Thuasne Sport
- zh Thuasne Sport 加强韧带护膝
- ar Thuasne Sport داعم أربطة الركبة المعزز

	S	M	L	XL
	43 - 46	46 - 49	49 - 52	52 - 55

fr	Circonférence à prendre 15 cm au-dessus du genou
en	Take the circumference 15 cm above knee
de	Umfang 15 cm über dem Knie messen
nl	Omtrek te meten 15 cm boven de knie
it	Circonferenza da misurare 15 cm al di sopra del ginocchio
es	Tomar la circunferencia situada 15 cm por encima de la rodilla
pt	Medir a circunferência 15 cm acima do joelho
da	Omkredsen skal måles 15 cm over knæet
fi	Mittaa ympärys 15 cm polven yläpuolelta
sv	Omkrets ska tas 15 cm ovanför knäet
el	Περίμετρος του μηρού όπως μετράται 15 εκ. πάνω από το γόνατο
cs	Obvod měřte 15 cm nad kolénem
pl	Obwód (15 cm powyżej kolana)
lv	Apkārtmērs jānosaka 15 cm virs ceļgala
lt	Apimtis matuojama 15 cm virš kelio
et	Ümbermõõt mõõta 15 cm põlvest kõrgemal
sl	Obseg stegna 15 cm nad kolénom
sk	Obvod sa meria 15 cm nad kolénom
hu	A körfogatot a térd felett 15 cm-t kell venni
bg	Размерът на обиколката, която да се вземе предвид е 15 cm над коляното
ro	Circumferința de prindere 15 cm sub genunchi
ru	Обхват ноги, измеренный на 15 см выше колена
hr	Opseg bedra potrebno je izmjeriti na 15 cm iznad koljena
zh	膝盖上方15厘米处量取15厘米周长
ar	المحيط الذي ينبغي أخذه 15 سم فوق الركبة

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en

THUASNE SPORT REINFORCED LIGAMENT KNEE BRACE

Description/Destination:

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 4 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating a light and soft knit at the back of the knee;
- 2 rigid hinged supports providing knee stability ⓑ;
- 1 anterior half-straps ⓒ and 2 posterior half-straps ⓐ, ⓑ (including a suspension strap ⓐ);
- a patellar insert to support patella ⓓ.

Composition:

Textile components: polyamide - polyester - elastan - polyurethane - thermoplastic polyurethan - synthetic rubber (SBR) - natural rubber. Rigid components: polyamide - polyoxymethylene.

Properties/Mode of action:

Stabilization of knee joint ligaments provided by the rigid hinged side uprights.

An anatomically-shaped knitting for even compression.

The brace is maintained on the leg thanks to the suspension strap ⓐ.

Staggered straps to prevent tourniquet effect.

Indications:

Preventive use for the knee ligament injuries (lateral ligaments).

Joint instability/laxity.

Knee pain and/or swelling management.

Sport activities resumption.

Contraindications:

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions:

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Contains natural latex.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the product in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches..).

Undesirable side-effect:

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application:

It is recommended to wear the device directly on the skin, unless contraindicated.

To avoid damage to the knit fabric, please make sure that the self-gripping pads are always fastened to the designated self-gripping surfaces when putting it on and taking it off.

- Slip the knee brace on the leg with your foot extended, making sure the patella is properly positioned at the centre of the patellar insert.
- First tighten and fasten the suspension strap ④ located above the calf.
- Tighten strap ③, the back strap over the thigh.
- Tighten strap ②, the front strap over the thigh.

If you feel the need to tighten the straps during an activity, make sure you tighten them following the steps indicated in the fitting instructions.

Care:

Product can be washed in accordance with the conditions shown on this notice and on the label. Do not machine wash the device more than 30 times. Close the self-fastening tabs before washing. Machine wash at 30°C (delicate programme). If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage:

Store at room temperature, preferably in the original packaging.

Disposal:

Dispose of in accordance with local regulations.

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- 1 framre halvband © och 2 bakre halvband (A), © (inklusive ådragningsband (A)).
Thuasne SAS - SIREN/RCS Nanterre 542 091 186