
THUASNE

Genu PROMASTER

D: Patellabandage mit verstärktem propriozeptiven Effekt
GB/USA: Patellar knee brace with reinforced proprioceptive effect
F: Genouillère à effet proprioceptif renforcé
I: Ginocchiera rotulea con effetto proprioceptivo rinforzato
E: Rodillera rotuliana con efecto proprioceptivo reforzado
P: Joelheira rotuliana com efeito proprioceptivo reforçado
NL: Knieverband voor de knieschijf met versterkte proprioceptieve werking
DK: Knæbind med forstærket proprioceptiv virkning
S: Knäskålsförband med förstärkt proprioceptiv verkan
FIN: Polvilumpion suojuksella varustettu polvituki, jolla on tehokas asento- ja liikeaistiin ohdistuva vaikutus
GR: Συνδεσμική επιγονατίδα με ενισχυμένη ιδιοδεκτική δράση
CZ: Čěšková kolenní ortéza se zesíleným proprioceptivním účinkem
PL: Orteza stawu kolanowego ze wzmocnionym efektem proprioceptywnym
EST: Põlvekedrabandaaž tugevdatud propriotseptiivse efektiga
LV: Ceļa kaula bandāža ar uzlabotu proprioceptīvo efektu
LT: Sutvirtintas kelio girtelės įtvaras su proprioceptiniu poveikiu
SK: Kolenná ortéza so zosilneným proprioceptívnym účinkom
SLO: Kolenski opornica za pogačico z okrepljenim proprioceptivnim učinkom
H: Patella védő térdrögzítő megerősített proprioceptív hatással
RO: Genunchieră rotuliană cu efect proprioceptiv pronunțat
BG: Пателарна колянна ортеза с подсилено proprioceptивно действие
HR: Bandaža za patelu (čšašicu koljena) s pojačanim proprioceptivnim učinkom

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XXS	35 – 38	13 ¾ - 15	25 – 28	9 ¾ - 11
XS	38 – 41	15 - 16	28 – 31	11 - 12 ¼
S	41 – 44	16 - 17 ¼	31 – 34	12 ¼ - 13 ½
M	44 – 47	17 ¼ - 18 ½	34 – 37	13 ¼ - 14 ⅔
M+	47 – 50	18 ½ - 19 ¾	34 – 37	13 ¼ - 14 ⅔
L	47 – 50	18 ½ - 19 ¾	37 – 40	14 ⅓ - 15 ¾
L+	50 – 53	19 ¾ - 21	37 – 40	14 ⅓ - 15 ¾
XL	50 – 53	19 ¾ - 21	40 – 43	15 ¾ - 17
XL+	53 – 56	21 - 22	40 – 43	15 ¾ - 17
XXL	53 – 56	21 - 22	43 – 46	17 - 18
XXL+	56 – 59	22 - 23 ¼	43 – 46	17 - 18

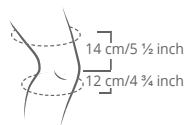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GB

USA

Patellar knee brace with reinforced proprioceptive effect

Purpose:

Genu **PROMASTER** is a compression bandage with patella pad for treating knee irritation. The bandage may only be used for knee treatment.

Properties:

- Brace made of elastic knit
- 3D knit at the patella
- Highly elastic knit for comfort of the popliteal fossa
- Patellar pad
- Integrated lateral reinforcements
- Latex-free

Indications:

Post-traumatic, postoperative and chronic irritation; Fibromyalgia and sprains of the lateral ligaments; Osteo- and rheumatoid arthritis; Mild joint instabilities; Patellar deficiencies

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the kneecap is properly centered in the central patellar ring. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Composition:

78 % PA (polyamide), 14 % EL (elastane), 5 % CO (cotton), 2 % PU (polyurethane), 1 % PES (polyester)

Explanation of the reinforced proprioceptive effect:

Orthoses in the **PROMASTER** range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH