
THUASNE®

Genu FS

- de Gebrauchsanweisung - Knieorthese zur Beeinflussung des Patellagleitweges
en Instructions for use - Knee orthosis for influencing the patella glide path
fr Consignes d'utilisation - Genouillère destinée à influencer la course rotulienne
it Istruzioni per l'uso - Ortesi per il ginocchio per il controllo dello scorrimento rotuleo
es Instrucciones de utilización - Ortesis de rodilla que actúa sobre el recorrido de la rótula
pt Instruções de uso - Ortótese para o joelho para a modificação do trajeto de deslize da rótula
nl Gebruiksaanwijzing - Knie-orthese voor de beïnvloeding van de patella-groeven
da Brugsanvisning - Knæortose til påvirkning af patellaens glidebane
sv Bruksanvisning - Knäortos för påverkan av patellas glidyta
fi Käyttöohje - Polvitukiside polvilumpion liikeradan korjaamiseen
el Οδηγίες χρήσης - Κηδεμόνας γόνατος για επενέργεια στην τροχιά ολίσθησης της επιγονατίδας
cs Návod k použití - Kolenní ortéza pro korekci pately
pl Sposób użytkowania - Orteza stawu kolanowego korygująca tor ruchu rzepki
et Kasutusjuhend - Põlveortoos põlvekedra liikumistee suunamiseks
lv Lietošanas instrukcija - Ceļgala ortoze ar ietekmi uz ceļa kaula slīdes kustību
lt Naudojimo instrukcija - Kelio įtvaras girnelės padėties fiksavimui
sk Návod na použitie - Kolenná ortéza na ovplyvnenie vedenia pately
sl Navodila za uporabo - Kolenska ortoza za vpliv na drsno pot pogačice
hu Használati utasítás - Térd ortézis a patella pályájának befolyásolásához
ro Instrucțiuni de utilizare - Orteză pentru genunchi pentru influențarea căii de alunecare a patelui
bg Инструкция за употреба - Колянна ортеза за повлияване на траекторията на плъзгане на пателата
hr Upute za uporabu - Ortoza za koljeno za modulaciju putanje patele

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XS	38 – 41	15 - 16	28 – 31	11 - 12 ¼
S	41 – 44	16 - 17 ¼	31 – 34	12 ¼ - 13 ½
M	44 – 47	17 ¼ - 18 ½	34 – 37	13 ¼ - 14 ¾
L	47 – 50	18 ½ - 19 ¾	37 – 40	14 ¾ - 15 ¾
XL	50 – 53	19 ¾ - 21	40 – 43	15 ¾ - 17
XXL	53 – 57	21 - 22 ½	43 – 46	17 - 18

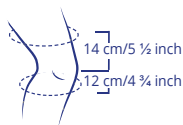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¹Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Veľkosť · Velikost · Méret · Mărimē · Размѐр · Veličina

²Thigh circumference · Tour de cuisse · Circonferenza coscia · Contorno del muslo · Perímetro da coxa · Dijbeenomvang · Lårbensomkreds · Lårvidd · Reiden ympärys · Περίμετρος μηρού · Vystouplost břícha · Obwód uda · Reie ümbermöö · Augšstilba apkārtmērs · Šlaunies apimtis · Vystúplost brucha · Obseg stegna · Combkörfogat · Circumferința coapsă · Обиколка на бедрата · Opseg natkoljenice

³Calf circumference · Tour de mollet · Circonferenza polpaccio · Contorno de la pantorrilla · Perímetro da barriga da perna · Kuitomvang · Lægomkreds · Vadmått · Pohkeen ympärys · Περίμετρος κνήμης · Obvod lýtky · Obwód łydki · Sääreümler-möö · Ikru apkārtmērs · Blauzdos apimtis · Obvod lýtky · Obseg stegna · Vádlikörfogát · Circumferința gambă · Обиколка на прасеца · Opseg listova

⁴Left or right · Droite ou gauche · Sinistro o destro · Izquierda o derecha · Esquerda ou direita · Links of rechts · Venstre eller højre · Vänster eller höger · Vasen tai oikea · Αριστερ. ή δεξ. · Vlevo nebo vpravo · Strona lewa lub prawa · Vasakul või paremal · Kreisā vai labā puse · Kairēje arba dešinėje · Vľavo alebo vpravo · Levo ali desno · Balos vagy jobbos · Stânga sau dreapta · лев или десен · Lijevo ili desno

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Knee orthosis for influencing the patella glide path

Purpose:

The Genu FS is a knee orthosis with traction system and patella pad for influencing and correcting the patella glide path. The orthosis may only be used for knee treatment.

Properties:

- Traction system for influencing and correcting the patella glide path
- Fast and easy correction adjustment
- Direct force transmission
- Compressive knit fabric
- Flexible knit fabric in the hollow of the knee
- Calf strap
- Latex-free

Indications:

- Femoro-patellar pain syndrome
- Patella alta
- Patellar lateralisation

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Directions for putting on the orthosis

Before you put it on:

- Please open the calf strap and loosen the patella traction system first.
- To avoid damage to the knit fabric, please make sure that the Velcro pads are always fastened to the designated Velcro surfaces when putting it on and taking it off.

Putting on the orthosis:

- Grasp the orthosis at the top.
- Turn the orthosis by approximately 90°. Step into the orthosis with your foot extended and then pull it up over your leg.
- Please make sure that the patella pad is placed correctly around the patella (= knee cap).

Adjusting the pressure of the traction system:

- Upper strap: To adjust the traction system individually, simply undo the Velcro fastening of the upper strap on the inside of the leg, apply slight tension to pull it towards the inside of your leg until the pressure is correct, then re-fasten the Velcro tab. The strap can be cut to suit if it is too long.
- Calf strap: First undo the Velcro fastening and loosen the strap. To adjust the pressure of the traction system, apply slight tension to pull the strap towards the

calf until the pressure is correct. The end of the Velcro tab can be fastened right on the strap. The strap can be cut to suit if it is too long.
Caution: To avoid constriction, please make sure that the strap is not pulled too tightly. Loosen it if required.

Before you take it off:

- Please open the calf strap and loosen the thigh strap. Then reach from below into the orthosis and pull it down and off.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH