THUASNE®

Malleo Dynastab® BOA®

Malleo Dynastab® BOA® Open

fr	Chevillère d'immobilisation stabilisatrice avec système de serrage BOA® Fit System.....	7
en	Stabilising ankle-immobilising brace with BOA® Fit System closure ...	8
de	Stabilisierende Sprunggelenkorthese zur Ruhigstellung mit BOA® Fit System zum Festziehen	9
nl	Stabiliserende immobilisatiebrace voor de enkel met BOA® Fit System-klemsysteem.....	11
it	Cavigliera d'immobilizzazione stabilizzatrice con sistema di chiusura BOA® Fit System.....	12
es	Tobillera de inmovilización estabilizadora con sistema de ajuste BOA® Fit System.....	14
pt	Artelheira estabilizadora de imobilização com sistema de aperto BOA® Fit System.....	15
da	Immobiliserende ankelstøttebind med BOA® Fit System til stramning.....	17
fi	Immobilisoiva vakauttava nilkkatuki, jossa on BOA® Fit System -kiristysjärjestelmä.....	18
sv	Stabiliserande och immobiliserande ankelortos med fästsystemet BOA® Fit System.....	20
el	Σταθεροποιητική επιστραγαλίδα ακινητοποίησης με σύστημα σύσφιξης BOA® Fit System.....	21
cs	Stabilizační ortéza na kotník se stahovacím systémem BOA® Fit System.....	23
pl	Stabilizująca orteza skokowo-stopowa z systemem dopasowania BOA® Fit System.....	24
lv	Stabilizējošs imobilizācijas potītes stiprinājums ar BOA® Fit System savilkšanas sistēmu.....	25
lt	Stabilizuojantis imobilizuojantis čiurnos įtvargas su „BOA® Fit System“ suveržimo sistema.....	27
et	BOA® Fit System kinnitussüsteemiga stabiliseeriv fikseeriv hüppeliigese ortoos.....	28
sl	Imobilizacijska opornica za stabilizacijo gležnja s sistemom zategovanja BOA® Fit System.....	30
sk	Imobilizačná členková ortéza s upínacím BOA® Fit Systémom.....	31
hu	Stabilizáló bokarögzítő BOA® Fit System szorítórendszerrel.....	32
bg	Обездвижващ стабилизатор за глезен със система за затягане BOA® Fit System.....	34
ro	Gleznieră stabilizatoare de imobilizare cu sistem de strângere BOA® Fit System.....	35
ru	Ортез стабилизирующий голеностопный с системой фиксации BOA® Fit System.....	37
hr	Steznik za stabilizaciju i imobilizaciju sa sustavom zatezanja BOA® Fit System.....	38
zh	带BOA® Fit System闭锁系统的稳定式护踝.....	40
ar	BOA® Fit System داعم كاحل لتحقيق الاستقرار والثيب مزود بنظام الشد.....	43

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzing - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegółowy użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

Malleo Dynastab® BOA®

	0	16 - 19 cm	6 ^{3/4} - 7 ^{1/2} "
	1	19 - 22 cm	7 ^{1/2} - 8 ^{3/4} "
	2	22 - 26 cm	8 ^{3/4} - 10 ^{1/4} "
	3	26 - 30 cm	10 ^{1/4} - 11 ^{3/4} "

Malleo Dynastab® BOA® Open

	1	19 - 22 cm	7 ^{1/2} - 8 ^{3/4} "
	2	22 - 26 cm	8 ^{3/4} - 10 ^{1/4} "
	3	26 - 30 cm	10 ^{1/4} - 11 ^{3/4} "



fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
ru	Окружность голени
hr	Opseg gležnja
zh	腳踝尺寸
ar	محيط الكاحل




fr	Immobilisation	Stabilisation	Proprioception
en	Immobilisation	Stabilisation	Proprioception
de	Ruhigstellung	Stabilisierung	Propriozeption
nl	Immobilisatie	Stabilisatie	Proprioceptie
it	Immobilizzazione	Stabilizzazione	Propriocezione
es	Inmovilización	Estabilización	Propiocepción
pt	Imobilização	Estabilização	Proprioceção
da	Støtte	Stabilisering	Proprioception
fi	Immobilisoi	Stabiloi	Proprioseptio
sv	Immobilisering	Stabilisering	Proprioception
el	Ακινητοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
cs	Znehybnění	Stabilizace	Propriocepce
pl	Unieruchomienie	Stabilizacja	Propriocepcja
lv	Imobilizācija	Stabilizēšana	Propriocepcija
lt	Imobilizavimas	Stabilizavimas	Propriocepcija
et	Liikumatuks muutmine	Stabiliseerimine	Propriotseptsoon
sl	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
sk	Imobilizácia	Stabilizácia	Propriocepčia
hu	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
bg	Обездвижване	Стабилизация	Проприоцепция
ro	Imobilizare	Stabilizare	Propriocepție
ru	Иммобилизация	Стабилизация	Проприоцепция
hr	Imobilizacija	Stabilizacija	Propriocepција
zh	固定	稳定	本体感受
ar	التثبيت	الاستقرار	استقبال الحس العميق

Malleo Dynastab® BOA®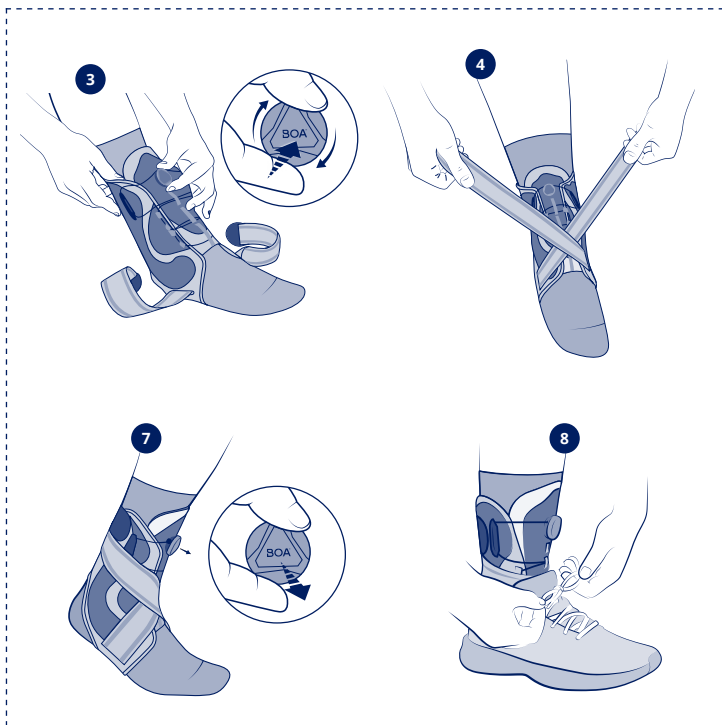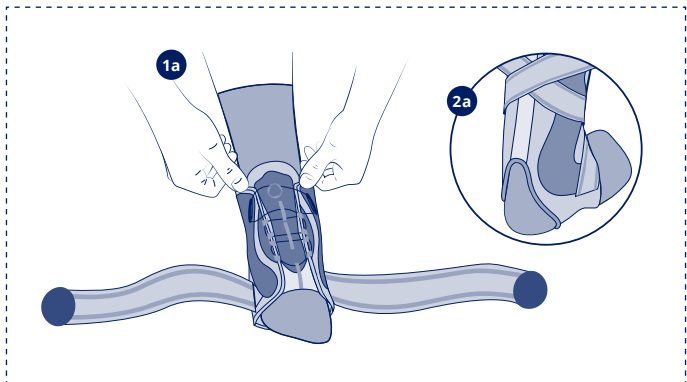

Malleo Dynastab® BOA® Open

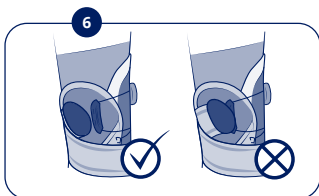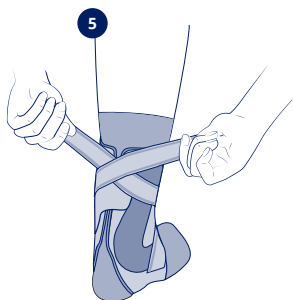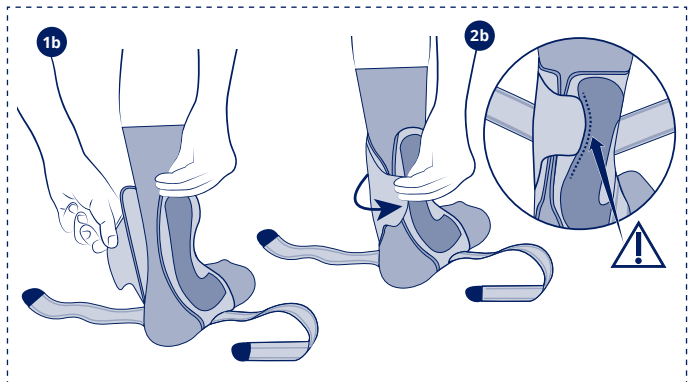fr	Appuyez pour enclencher	Tournez pour serrer	Tirez pour desserrer rapidement
en	Push in to engage	Turn to tighten	Pull up for quick release
de	Zum Einrasten drücken	Zum Festziehen drehen	Zum Öffnen ziehen
nl	Indrukken om aan te draaien	Draaien om aan te spannen	Optrekken om snel los te maken
it	Spingere per fissare	Girare per stringere	Tirare per rilasciare
es	Empuje para activar	Gire para apretar	Tire para liberar rápidamente
pt	Premir para acionar	Rodar para apertar	Puxar para cima para soltar rapidamente
da	Skub ind for at fastgøre	Drej for at stramme	Træk op for hurtigt at løsne
fi	Kytke painamalla	Kiristä kiertämällä	Vapauta nopeasti vetämällä
sv	Tryck in för att sätta fast	Vrid för att dra åt	Dra uppåt till för att lossa snabbt
el	Πιέστε προς τα μέσα για ενεργοποίηση	Στρέψτε για σύσφιξη	Τραβήξτε προς τα έξω για γρήγορη απελευθέρωση
cs	Stisknutím zacvakněte	Otočením utáhněte	Zvednutím povolte
pl	Wcisnąć, aby wprowadzić	Przekręcić, aby zacisnąć	Pociągnąć w górę, aby szybko zwolnić
lv	Nospiediet, lai nofiksētu	Pagrieziet, lai savilktu	Pavelciet, lai ātri atbrīvotu
lt	Paspauskite, kad sujungtumėte	Pasukite, kad suveržtumėte	Patraukite, kad greitai atlaisvintumėte
et	Sulgemiseks vajutage	Pingutamiseks keerake	Kiireks lahti tegemiseks tõmmake
sl	Pritisnite, da vklopite	Vrtite, da zategnete	Povlecite gor, da se hitro odklopi
sk	Zatlačením zaistite	Otočením utiahnite	Potiahnutím rýchlo uvoľnite
hu	Rögzítéshez nyomja meg	Szorításához fordítsa el	Gyors kioldáshoz húzza ki
bg	Натиснете, за да поставите	Завъртете, за да затегнете	Издърпайте нагоре за бързо освобождаване
ro	Apăsați pentru a cupla	Rotiți pentru a strânge	Trageți în afară pentru a elibera rapid
ru	Нажмите для установки в рабочее состояние	Поверните, чтобы затянуть	Потяните, чтобы растянуть
hr	Pritisnite za zaključavanje	Okrenite za zatezanje	Povucite za brzo otpuštanje
zh	向下压启用	旋转拧紧	向上拔起快速释放
ar	اضغط للتركيب	أدر للشد	اسحب لفكه بسرعة

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Conserver cette notice.

en

STABILISING ANKLE-IMMOBILISING BRACE WITH BOA® FIT SYSTEM CLOSURE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.
This device can be used when resuming sports activities.

Composition

Textile parts: polychloroprene, polyamide, elastane, polyester, polyethylene, polyurethane.

Elastic straps: polyester, elastane.

Self-fastening tabs: polyamide.

Plastic parts: polyamide 66.

Side reinforcements: stainless steel.

Properties/Mode of action

Stabilising ankle brace to provide immobilisation, incorporating rigid frame and elasticated webs to give stability and immobilisation to the ankle.

BOA® Fit System for tightening adjustment.

A slim product that will fit easily inside a shoe.

This is a bilateral product.

Indications

Moderate to severe ankle sprain.

Chronic ankle instability.

Ankle pain and oedema.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene and performance reasons, do not re-use the product for another patient.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the device in a medical imaging machine.

Do not wear the device when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral straps. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

We recommend that the product should be worn inside a shoe by daytime whilst walking, and by night for protection in order to prevent and limit sudden movements.

Before putting the ankle brace on, make sure the laces are completely unfastened.

Fitting the ankle brace

Malleo Dynastab® BOA®

Slide the foot inside the ankle brace. Ensure that the heel is resting inside the opening provided at the rear of the product for this purpose **1b** **2a**.

Malleo Dynastab® BOA® Open

Slide the foot inside the stirrup, move the foot forward until you feel the lateral stays positioned right behind the malleoli (the curve of the stays should follow the malleoli shape). Close the self-fastener at the back of the brace without tightening it too much (tightening adjustment will be done thanks to the BOA® Fit System) **1b** **2b**.

Take care to cover the hook-and-loop with the closing tab without protruding over the black part.

Fastening the ankle brace

Tighten the ankle brace using the BOA® Fit System closure system; to do this, push the wheel in until you hear a 'click', then turn clockwise until the correct level of support is obtained **3**.

Fitting the straps

Take a strap in each hand, then pass them over each other across the upper foot **4**.

Wrap around the ankle, cross the two straps over again around the rear of the heel, and attach the self-fastening tabs to the upper portion of the ankle-brace fabric **5**.

Verify the secure grip of the self-fastening tabs over the fabric area by pressing on the self-fastening part. Do not place the self-fastening tabs over the black plastic portion **6**.

Releasing the ankle brace:

Pull on the wheel until you hear the 'click' to unfasten the laces **7**.

Fitting inside a shoe:

Once the ankle brace is in position, slide the foot carefully inside a closed shoe and fasten the shoe's closure mechanism **8**.

In the event of any discomfort at the ankle-bone area, it is possible to change the position of the stays: Detach the self-fastening tabs from the black fabric area, take hold of the rigid metal part and remove it. Slide the stays above or below the stitching located either side of the ankle brace **9**.

Care/maintenance

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label.

Close the self-fastening tabs and close the product (BOA® Fit System) before washing. If possible, use a washing net.

Machine washable at 40 °C. It is not recommended to machine wash the device more than 24 times (average: two wash per month). Make sure to remove the lateral stays before washing. Replace in the original location before next use. Do not dry clean. Do not tumble-dry. Do not iron. Dry in a ventilated area, away from sources of heat.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2054801 (2024-07)