THUASNE

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Paddings kit for the Offloading rigid knee brace Rebel Reliever®.

Description/Destination

The kit is intended to be used with the Rebel Reliever® Offloading rigid knee brace only to replace the inner paddings when necessary.

Carefully read the indications, contraindications, precautions and undesirable side-effects of the Rebel Reliever® knee brace instructions for use.

The Paddings kit includes:

- 1 femoral inner padding Ⓐ (size in correlation with the brace femoral shell size),
- 1 tibial inner padding Ⓑ (size in correlation with the brace tibial shell size).

Composition

Textile components: elastane - polyamide - silicone - polyethylene - poly-methyl methacrylate - ethylene vinyl acetate.

Paddings application:

To remove the used paddings, simply pull them away from the frame of the brace.

Then, fix on the interior side of the brace's frame the new paddings on the self-fastening surfaces.

Once the adjustment of the knee brace is finalized, it is possible to trim any excess padding at the lower end of the pads, just above both hinges.

Thuasne cannot be held responsible for undesirable effects or injuries resulting from any unsupervised or inadapted adjustments.

Care/Maintenance

Refer to the maintenance instructions of the Rebel Reliever® offloading rigid knee brace.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186